GIMA

**LARINGOSCOPI CONVENZIONALI
CONVENTIONAL LARYNGOSCOPES
LARYNGOSCOPES CONVENTIONNELS
KONVENTIONELLE LARYNGOSKOPE
LARINGOSCOPIOS CONVENCIONALES
LARINGOSCÓPIOS TRADICIONAIS
ΣΥΜΒΑΤΙΚΑ ΛΑΡΥΓΓΟΣΚΟΠΙΑ**

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ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

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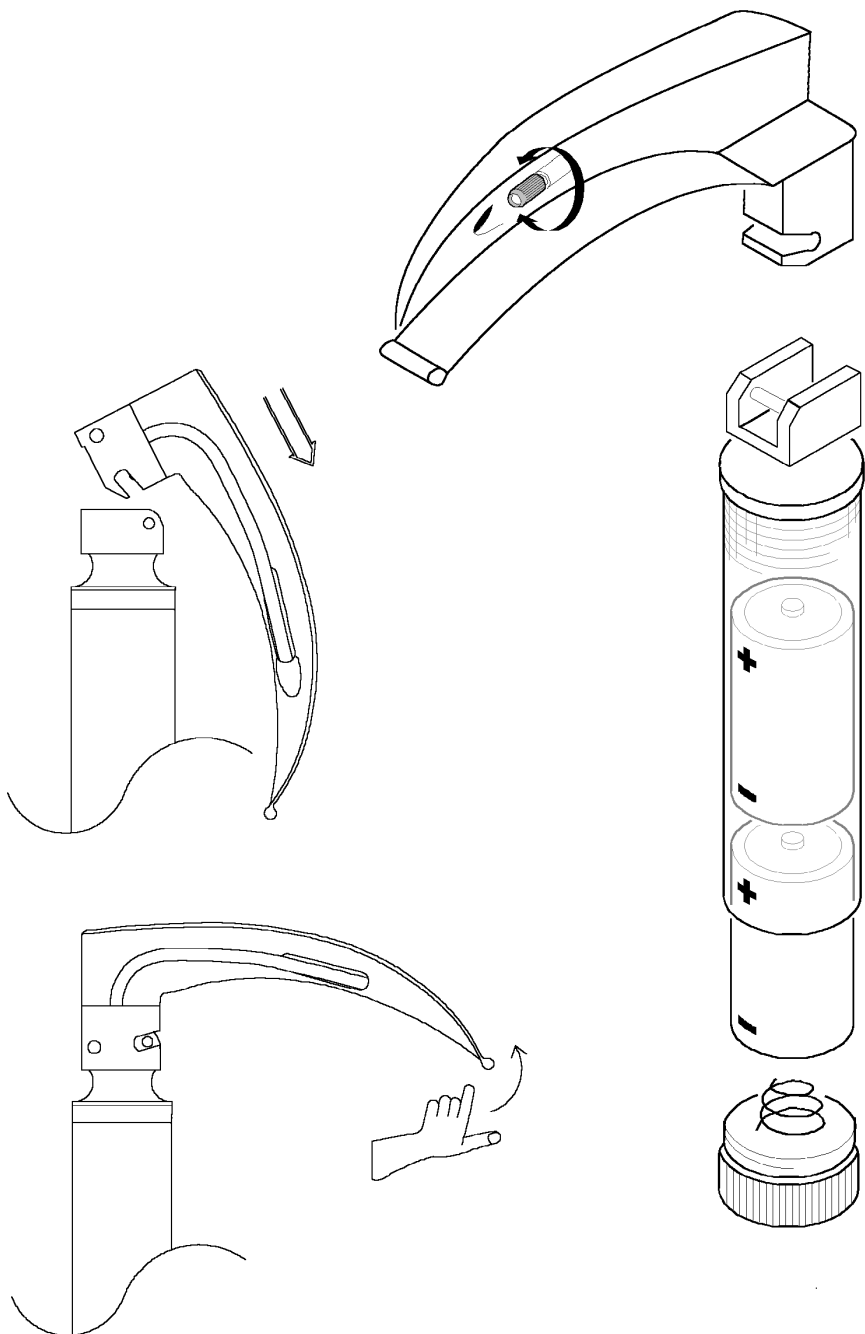
INTERNATIONAL: Tel. ++39 02 953854209 - Fax ++39 02 95380056

E-mail: export@gimaitaly.com - www.gimaitaly.com

DO NOT scrape or use abrasive material to clean the plated contacts on the handle because they might be damaged.

DO NOT plunge the handles in cleansing agents or solutions which are not proper for this use.

DO NOT put handles in autoclave without removing batteries!

GUARANTEE

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

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
Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.

The guarantee will be invalidated if the registration number has been removed, cancelled or altered.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

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