MANUEL D'UTILISATION / USER'S MANUAL / دليل الاستخدام

Lampe d'Examen LED Réglable
Adjustable LED Examination Lamp
مصباح فحص LED قابل للضبط

REF OSLAMP01

REF OSLAMP02

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 CELAB - Via Maira snc 04100 Latina - Italy

1. INTRODUCTION

The Holtex LED examination lamp is designed to provide selective and partial lighting during a medical examination and minor operations, mainly medical, hospital outpatient, or dental care.

Characteristics: light, flexible, easy to manoeuvre, practical operation.

Components: lamp head, goose neck flexible, control box with variator, metal base in caster wheels, support tube and power lead.

2. CHARACTERISTICS

LED technology without IR (Infrared): it reduces the sense of heat for the patient and the doctor. Tissue does not dry out as quickly during the operation under very weak light radiation.

Positioning and long-range: The articulated arm makes it easy to move and aim the light to avoid blind spots.

Variable illumination: The variator on the control box reduces or increases luminous intensity.

Energy-saving: The service life of a LED is 40 times longer than that of a halogen lamp. Low-energy: 1/3 less than a conventional halogen system.

Scope: Close-up examination, over the bed, intensive care, recovery rooms, dermatology, caring for the elderly, gynaecology, ophthalmology, ENT, minor surgery, etc.

3. TECHNICAL DATA

Lighting

- Light source: 3W LED
- Number of bulbs: 1 LED
- Intensity: D 30 cm = 80,000 lx, and D 50 cm = 35,000 lx
- Light field diameter: D 30 cm = 5 cm
- Colour temperature: 5000-6000K
- Control system: Luminosity adjusted by a variator/dimmer

Structure

- Single lamp head with On/Off handle
- Lamp head diameter: 7,6 cmm
- Lamp head length: 11 cm
- Flexible arm: Goose neck length: 76 cm, no blind spots

Power supply

Voltage: 100V-260V 50/60Hz

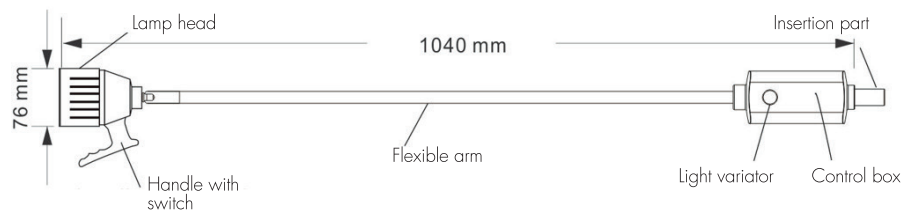
Power cord: 2.5 metres long

Presentation


Detachable mobile metal base, or fixing accessories sold separately: wall, table, rail clamp. The lamp is available in black or white.

4. PRODUCT DESCRIPTION

Top part of the lamp

Mobile base

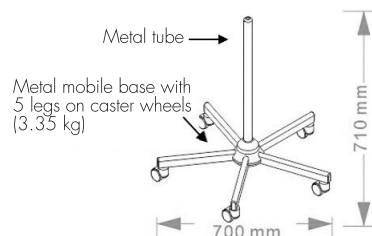Fixing accessories

5. SETTING UP AND ADJUSTING THE LAMP

- Open the parcel and check that everything is in order. If there is a problem, contact the manufacturer or distributor
- Remove the packaging from the lamp,
- Fit the casters on the mobile base then insert the big end of the support's metal tube into the mobile base.
- Connect the lamp to the power source and switch on the light.
- Adjust luminosity with the variator on the control box and switch the light on and off with the handle.
- Adjust the position of the lamp with its flexible arm and rotating head BUT be careful not to force it – do not exceed the angle limit.
- To install the optional wall mounting: drill holes in the wall then insert the spring-loaded screw and screw in the clamp.

CAUTION! Do not plug the lamp into a power extension block or extension cable. This could cause overheating and a fire.

6. MAINTENANCE AND SAFETY INSTRUCTIONS

PLEASE READ AND FOLLOW THE SAFETY INSTRUCTIONS

DO NOT LOOK DIRECTLY AT THE LEDs, whatever their intensity. LED light is intense and concentrated; risk of lesions. Do not aim the lamp directly at the eyes of a person or animal.

- Use the product only as specified in the manual. Improper use may damage the product or its environment.
- Keep the user manual and read it if the need arises.
- Dismantling or altering the product affects its safety. Caution, risk of injury.
- Never try to repair the product yourself (apart from changing the bulb with the kit provided).
- Handle the product with care; you may damage it if you knock it or drop it even from a low height.
- Never immerse the lamp in water or any other liquid.
- Do not turn on the lamp if it or its connecting cable are visibly damaged.
- The lamp's power cord cannot be replaced. If it is damaged, the lamp must be destroyed.

Before cleaning the lamp, remove the electric cord. Clean its surface at regular intervals with a damp cotton cloth. Liquid detergents are not recommended.

When replacing a damaged bulb, wait for it to cool down. Unscrew the wall fitting clockwise and use the set of tools and screws for Holtex LED examination lamps (sold separately).

7. USER ENVIRONMENT

This equipment can only be used indoors. The Holtex examination lamp must be stored or used in an enclosed space with relative humidity lower than 70%, a temperature between 10 and 30°C and an atmospheric pressure between 860 hPa and 1060 hPa.

8. AFTER-SALES

Thank you for purchasing a Holtex product. This product is guaranteed for 1 year from the date of purchase, provided it is used correctly, in accordance with the user's manual.

During this guarantee period, and in the case of a normal use of the product, Holtex will repair or replace the defective product (apart from the LED bulb) without billing parts or labour.

The manufacturer accepts no liability for material or non-material damage or loss. The guarantee is voided in the event of improper use and the guarantee does not cover:

1. Transport costs and risks.
2. The cost of repairs and/or faults resulting from repairs done by anyone other than Holtex After-Sales staff.
3. The cost resulting from non-acceptance of a complaint (such costs will be billed).
4. If the product's markings such as the model number or date of manufacture have been changed, removed or moved or are illegible, maintenance is not free of charge.

For the addresses, refer to the packaging or product documentation or your specialist retailer.

Repair or replacement under guarantee does not extend or renew the guarantee period.

The guarantee only applies if the complete product is returned along with the bill or original sales receipt issued in the name of the consumer by the retailer, who must have filled in the After-Sales form available for download at www.holtex.fr.

9. ENVIRONMENTAL PROTECTION

Help protect the environment and your safety; throw away the packaging in containers provided for the purpose and recycle your electrical products. Do not throw away electrical and electronic appliances with household waste. Take them to your local recycling centre.

Made in China.

Imported and distributed by HOLTEx – 30 rue Jean de Guiramand – 13290 Aix-en-Provence – France.

 Shantou Easywell technologies Co., Ltd - 6th Floor H5 BLD, NO 16 Liangjiang Rd, Shantou - PRC

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